

PEG-J Tube Care


Please contact Dr. Don Duerksen with any concerns

Changing The Connectors

Practical Handling of Duodopa™ Changing the Connector, Freka®

Equipment

Scissors, two haemostatic forceps (or one haemostatic forceps and one tissue forceps), compress, gloves and Freka® connectors for PEG FR 15/9.

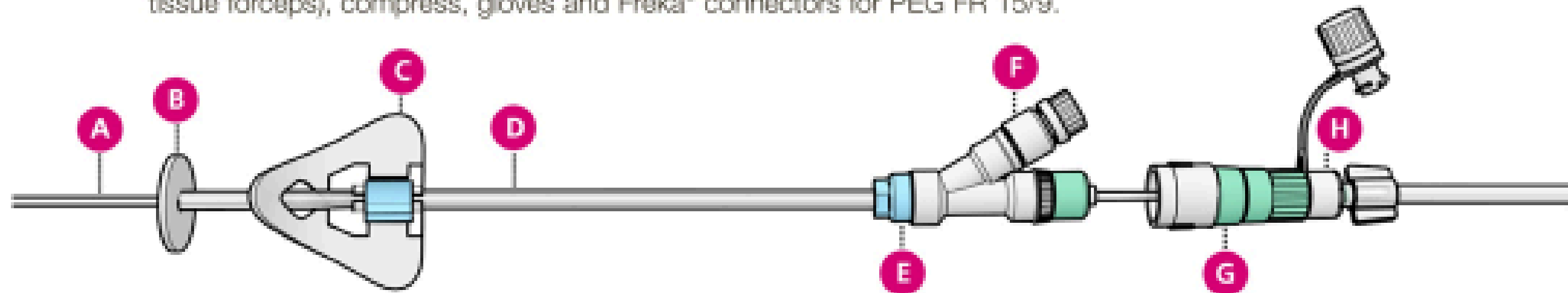


Freka PEG/J tube

A. Intestinal tube B. Internal retention plate C. External retention plate D. PEG tube
E. Fixation screw F. Y-connector G. Click connector H. Luer Lock connector


Reference: Instruction for use, Freka® PEG Set Gastric FR 15 / 20, Int. Instruction for use, Freka® Intestinal Tube FR 9 for PEG 15 / 20, Int. Instruction for Use, Changing the Connector (F1).

Practical Handling of Duodopa™ Changing the Connector, Freka®


Remove the Old Connector

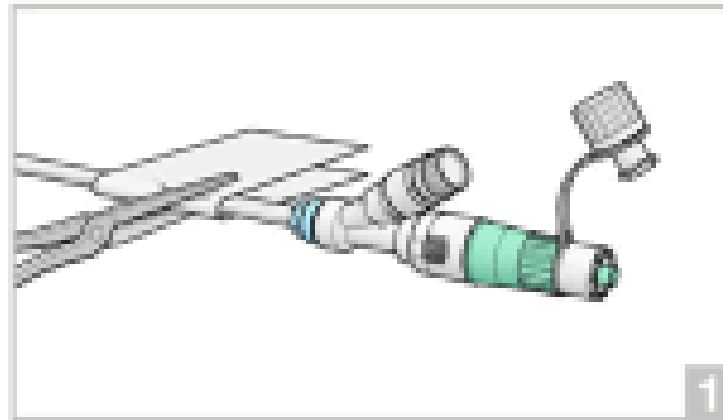

Fix the intestinal tube by locking the haemostatic forceps and the compress tightly on the PEG.

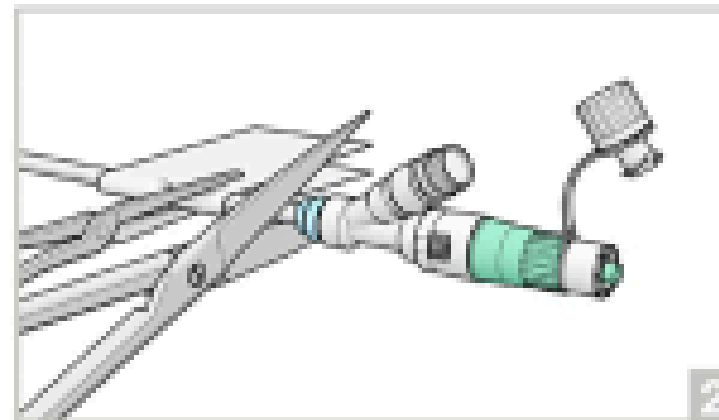

Make a straight cut through the PEG tube and intestinal tube close to the old connector.

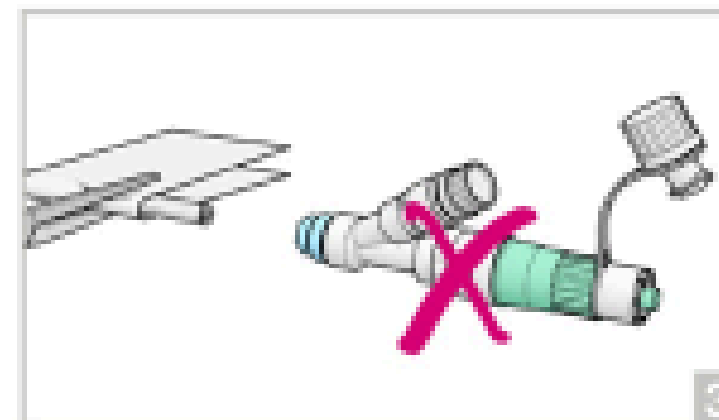


Discard the old connector.

Reference: Instruction for use, Freka® PEG Set Gastric FR 15 / 20, Int. Instruction for use, Freka® Intestinal Tube FR 9 for PEG 15 / 20, Int. Instruction for Use, Changing the Connector (F1).


Practical Handling of Duodopa™

Changing the Connector, Freka®


Prepare the Intestinal Tube

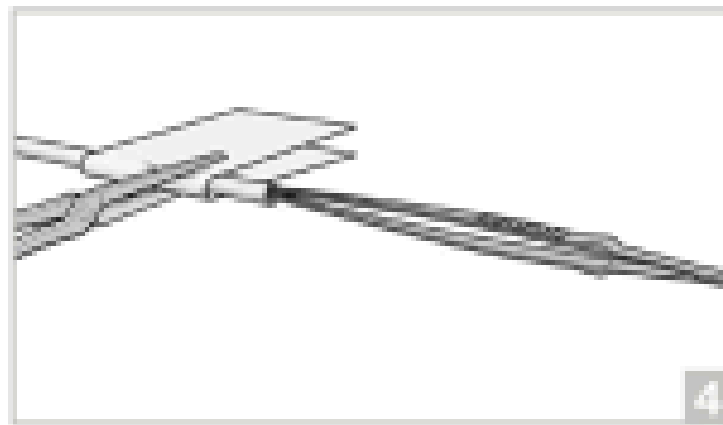

Grip the intestinal tube with the second tissue forceps (or another suitable forceps).

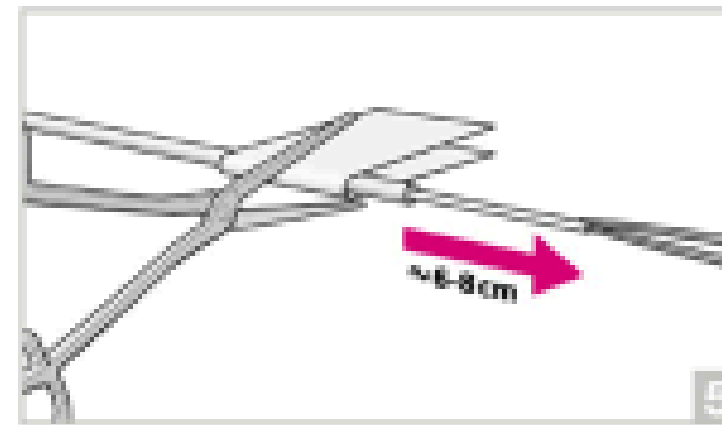

Carefully release the haemostatic forceps on the PEG and pull the intestinal tube out by approximately 6–8 cm.

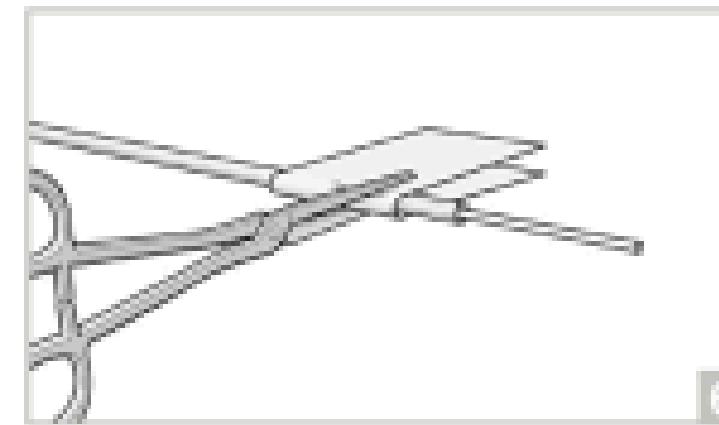


Place and lock the haemostatic forceps and the compress back on the PEG to fix the intestinal tube in position.

Reference: Instruction for use, Freka® PEG Set Gastric FR 15 / 20, version 47751531/00 Int. Instruction for use, Freka® Intestinal Tube FR 9 for PEG 15 / 20, version 15 0615411/5 Int. Instruction for Use, Changing the Connector (F1).


Practical Handling of Duodopa™

Changing the Connector, Freka®


Assemble the New Connector

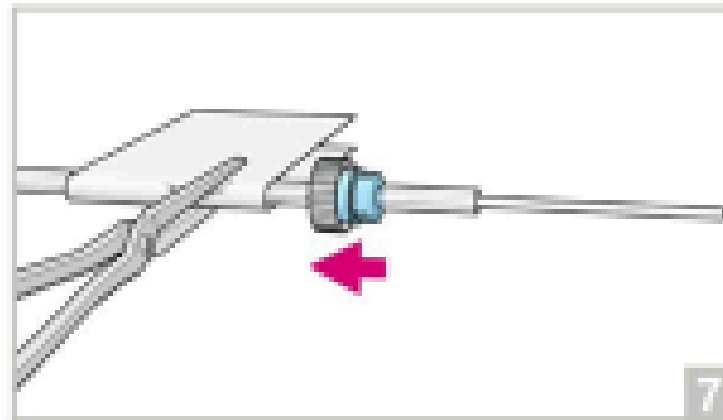

Push the blue and white fixation screw onto the PEG tube.

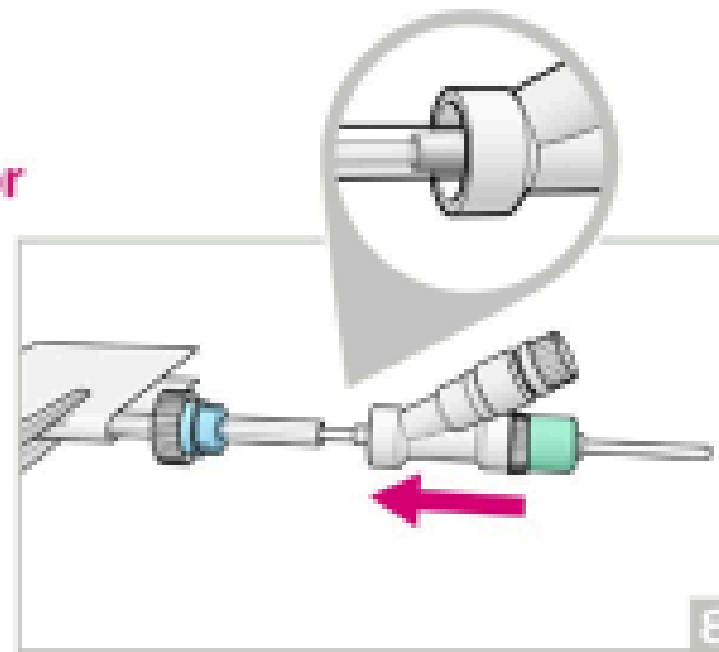

Push the white pin of the Y-connector onto the PEG tube as deep as possible. Before assembly, make sure the parts are dry.

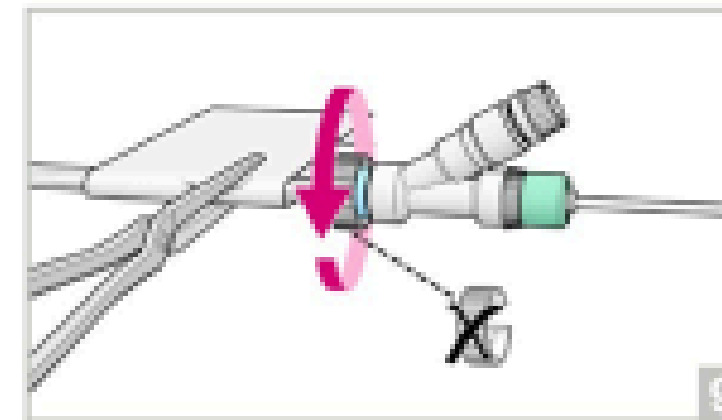


Screw the fixation screw onto the Y-connector all the way through. Thereafter pull off the screwing aid (outer white ring).

Reference: Instruction for use, Freka® PEG Set Gastric FR 15 / 20, version 47751531/00 Int. Instruction for use, Freka® Intestinal Tube FR 9 for PEG 15 / 20, version 15 0615411/5 Int. Instruction for Use, Changing the Connector (F1).


Practical Handling of Duodopa™

Changing the Connector, Freka®


Assemble the New Connector

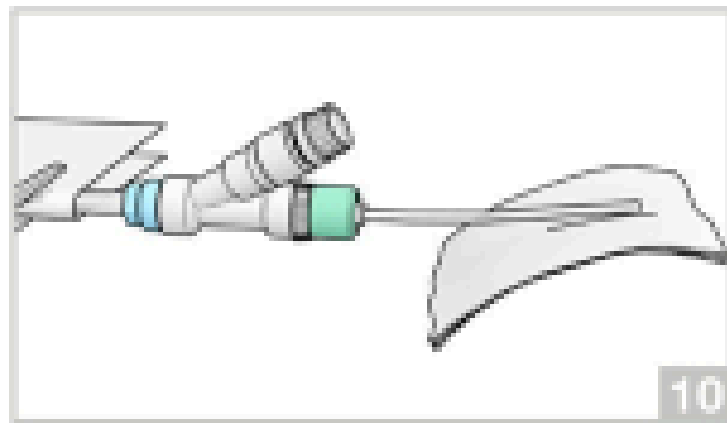


Dry the intestinal tube.

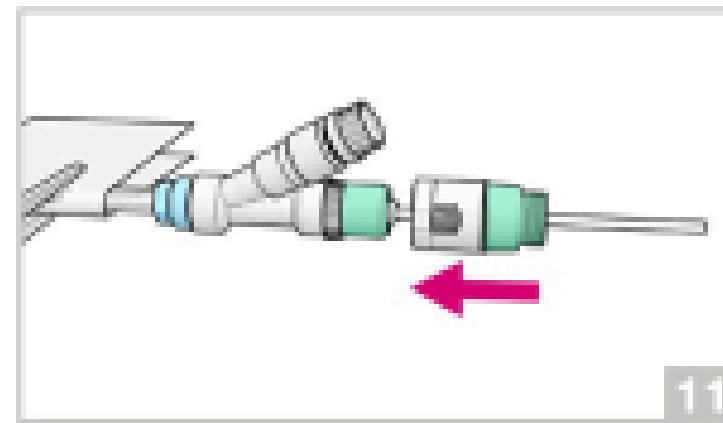

Push the green/white click connector onto the intestinal tube, leaving a small gap between the Y-connector and the click connector.

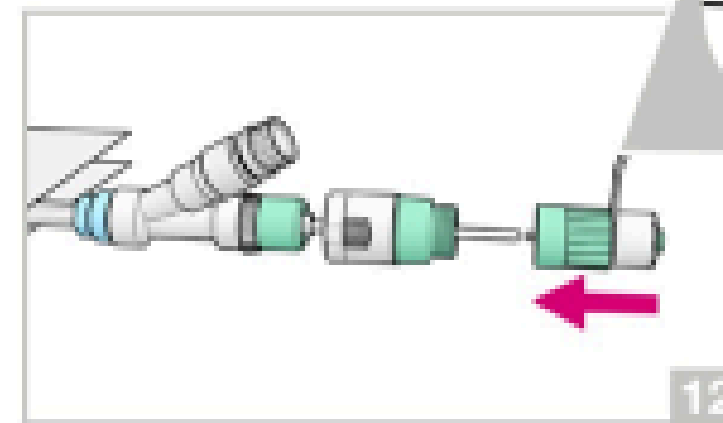

Make sure the metal pin is dry. Push the Luer Lock connector all the way into the intestinal tube.

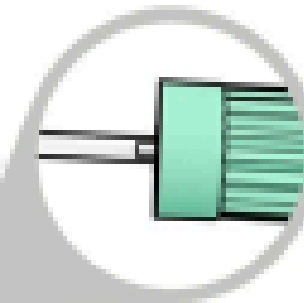


Reference: Instruction for use, Freka® PEG Set Gastric FR 15 / 20, Int. Instruction for use, Freka® Intestinal Tube FR 9 for PEG 15 / 20, Int. Instruction for Use, Changing the Connector (F1).


Practical Handling of Duodopa:™

Changing the Connector, Freka®


Assemble the New Connector

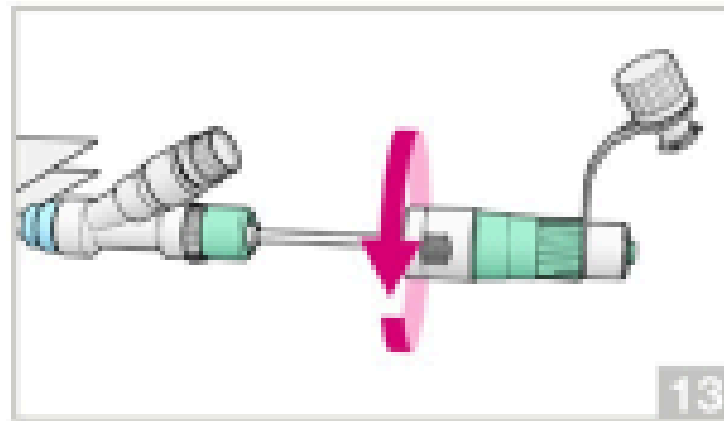

Screw the click connector tightly onto the Luer Lock connector.
Do not twist the tube.

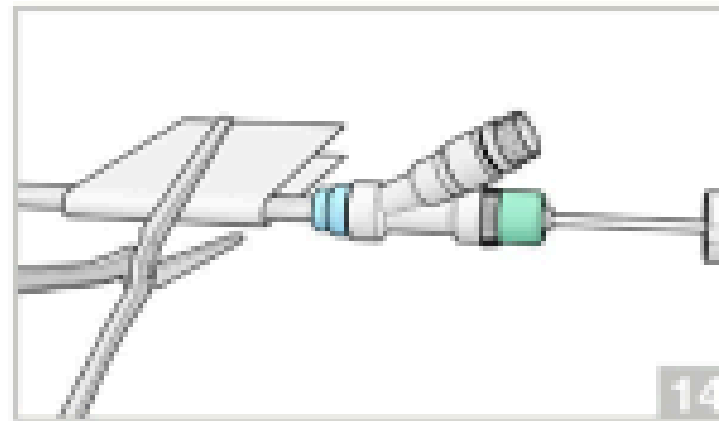

Release the forceps.

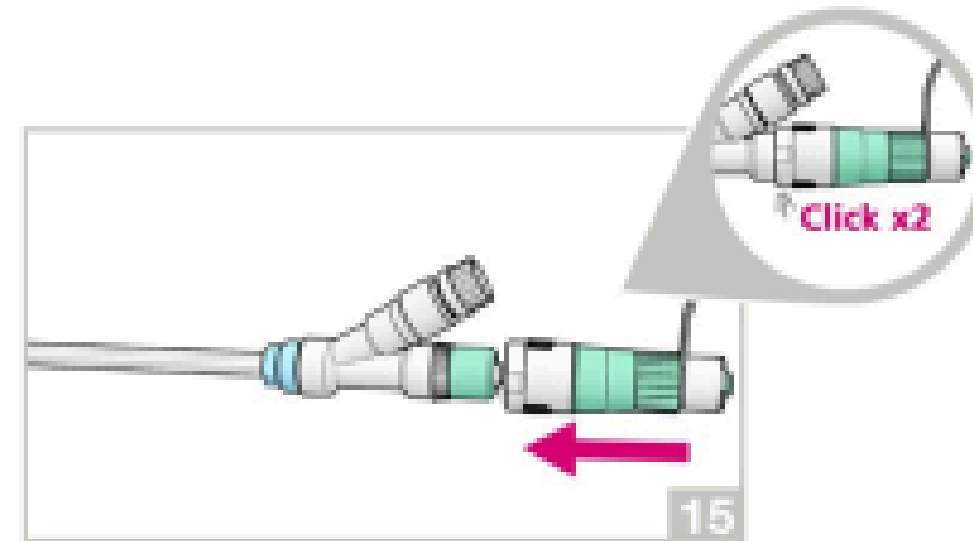

Push the two parts together firmly until a double click is heard, without twisting the tube. **Orientate the grip markers according to the image.**

Reference: Instruction for use, Freka® PEG Set Gastric FR 15 / 20, Int. Instruction for use, Freka® Intestinal Tube FR 9 for PEG 15 / 20, Int. Instruction for Use, Changing the Connector (F1).

Changing the Triangular Retention Plate

- **Unlock the retention plate from the PEG-J tubing**
- **Cut the old retention plate off**
- **With new retention plate, stretch and pull over the Y-Connectors**
- **Move to 0.5-1cm away from skin and lock tab**

Please Note:

Blue connectors & Retention Plate are for regular tubes

Purple connectors & Retention Plate are for Feeding tube size